

Purpose

Auto-laterality is a built-in device function wherein the software can detect which eye is being imaged. This document provides guidance to turn this function on for your *California rg* device.

NOTE: Full operating instructions and device warning and cautionary messaging are provided in the Instructions for Use (IFU). Please review the IFU (G-108025) prior to operating the device. The IFU can be found at optos.com/IFU. Scan the QR Code below for direct access:





Contents

Purpose.....	1
Enabling Auto-Laterality.....	1

Enabling Auto-Laterality


Note: Your device must have to Nurburg 2.4.4 version or later for auto-laterality to be enabled.

- 1 | Log into the device tablet using an account with Admin permissions.
- 2 | Select **To admin** in the upper right corner of the screen.
- 3 | Select **Configuration** menu option.
- 4 | Locate **Enable auto-laterality** and switch to **On**.

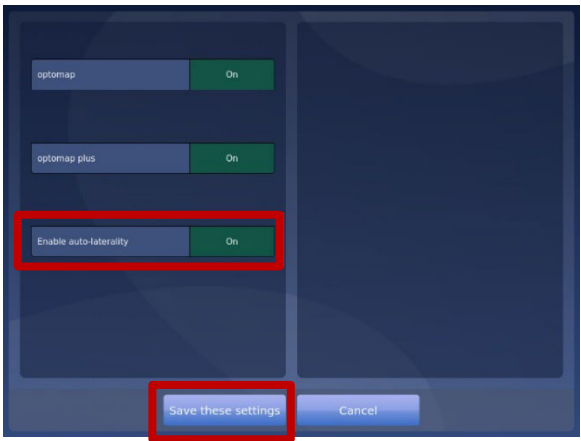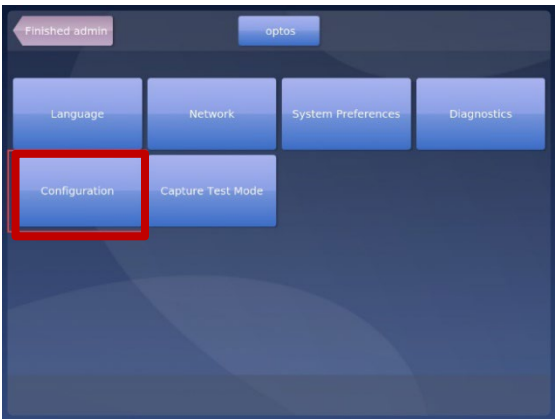


5 | Select **Save these settings** to retain your changes.

6 | Select **System Preferences**.

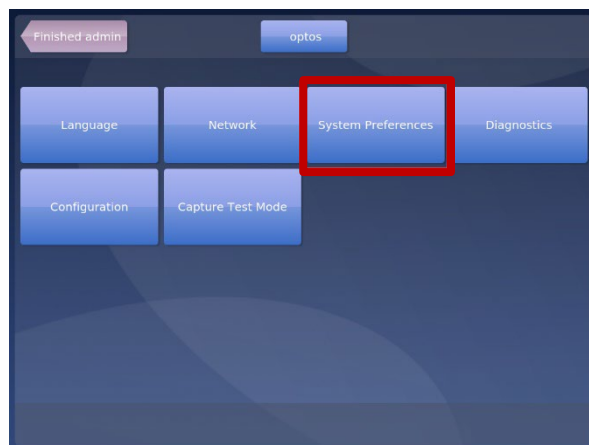

7 | Locate **Enable auto-laterality** and switch to **On**.

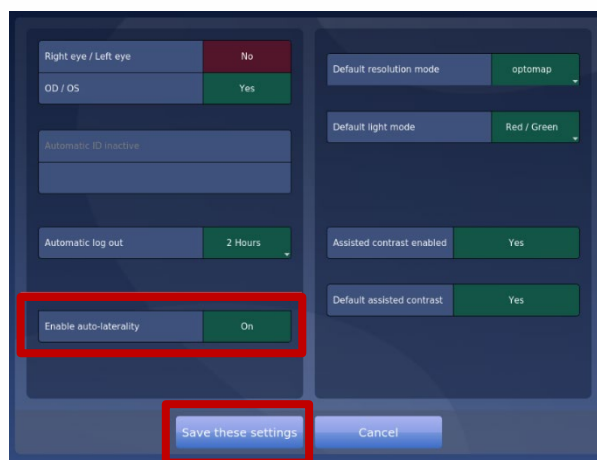

8 | Select **Save these settings** to retain your changes.

9 | Select **Finished admin** to complete and return to the patient search screen.

