

## General Device Information Tips for Daytona and UWF Primary



#### **Purpose**

This document provides guidance on how to capture quality images and tips on eye steering and lid holding techniques on *Daytona and* UWF Primary devices.

NOTE: Full operating instructions and device warning and cautionary messaging are provided in the Instructions for Use (IFU). Please review the IFU prior to operating the device. The IFU are found at optos.com/IFU. Scan the QR Code below for direct access:



#### **Contents**

Purpose	1
Initial Patient Positioning	1
General Tips	
Eye Steering Tips	
, J , Lid Holdina Tips	

#### **Initial Patient Positioning**

1 | Instruct the patient to sit up straight and move the chair forward so that they are approximately 6 inches from the device, looking straight ahead.



2 | Adjust the table height so that the tip of the nose is parallel with the top of the aperture.

Page **1** of **3** G-110008 / 4



# General Device Information Tips for Daytona and UWF Primary



3 | Instruct the patient to turn their head slightly to the side, with their nose on the outside of the face pad and slowly follow the blue light in towards the device, keeping the full target in their view until the target turns green.



- 4 | Confirm the eye is visible on the tablet with a green target and centered on the crosshairs. If the eye is not visible; ask the patient to sit back and repeat the above step.
- 5 | With the patient in correct alignment, instruct them to hold both eyes wide open and advise that there will be a bright flash of light as the image is captured.
- 6 | Tap the live eye viewport on the tablet to capture the image.

Page **2** of **3** G-110008 / 4



## General Device Information Tips for Daytona and UWF Primary



### **General Tips**

- The patient's head should be turned slightly so their nose is not pushed against the facepad.
- Instruct the patient to raise their eyebrows and open both eyes as wide as they can to minimize the presence of lids and lash in the image.
  - o If the patient has a pronounced eyebrow, ask them to move their cheek closer to the device.
  - o If the patient cannot locate the blue target, instruct them to sit back, close the eye not being imaged, follow the target in, and then open both eyes wide.
- If the patient has an IOL or very small pupils, image while the target is slightly in the red.
- If the patient is dilated, image while the target is slightly in the blue.
- If the patient had a YAG, image while the target is slightly in the red and make sure to be centered in the unobstructed area.

### **Eye Steering Tips**

NOTE: Eye Steering is not available on the UWF Primary Device.

- Instruct the patient to follow the target with their eyes.
- For inferior images, instruct the patient to raise their chin up slightly to see the target. Lift the upper lid if necessary, when you are ready to capture to minimize lid and lash on the image.
- For superior images, instruct the patient to move their chin down to see the target.
- Adjustment of the table height may be required to center the pupil in the crosshairs for superior and inferior gazes.

### **Lid Holding Tips**

Please consult with your office leadership on the preferred method for lifting lids.

- Use a long cotton swab or your index finger to hold up the lid right below the brow bone and closer towards the nose.
  - o If you hold the lid toward the canthus, you will raise the lower lid.
  - o If you hold the lid too close to the eyelash margin, your finger will show in the image.
  - Some patients may require holding both eyelids.
- Tape the eyelid per your physicians' instructions.



