

General Device Information

System Reboot



Purpose

This document provides guidance on the process to properly reboot your Optos® device and PC.

NOTE: Full operating instructions and device warning and cautionary messaging are provided in the Instructions for Use (IFU). Please review the IFU guides prior to operating the device. The IFU are found at optos.com/IFU. Scan the QR Code below for direct access:



Contents

Purpose.....	1
System Reboot.....	1

System Reboot

A reboot of the device is recommended weekly. Always ensure to follow the process outlined in this document.

- 1 | From the device tablet, select the **Shut Down** button in the top left corner.



- 2 | Turn off the device by pressing the on/off switch on the back of the device to the off position.
- 3 | Reboot the Optos PC by selecting the **Windows icon** on the bottom left of the screen. Select **Power Options**, and **Restart**.



- 4 | When the Optos PC has restarted, log in using the credentials set for your site.
- 5 | Turn the device on by pressing the on/off switch on the back of the device to the on position.

