

# OptosAdvance™ Changing Laterality of an Image



## Purpose

This document will provide guidance on how to change the laterality of an image if it was labeled incorrectly during capture.

NOTE: Full operating instructions are provided in the **Help** section of OptosAdvance. Once logged into the software, select the **Help** icon from the toolbar to search and browse guidance topics.


## Contents

Purpose.....	1
Changing Laterality .....	1

## Changing Laterality

You must have the correct privileges set in OptosAdvance user account to edit patient studies.

- 1 | Login to OptosAdvance.
- 2 | Open the desired Patient Study.
- 3 | Select the incorrectly labeled image; a blue box will appear around the viewport.
- 4 | Right-click your mouse and select **Edit Image Header**.

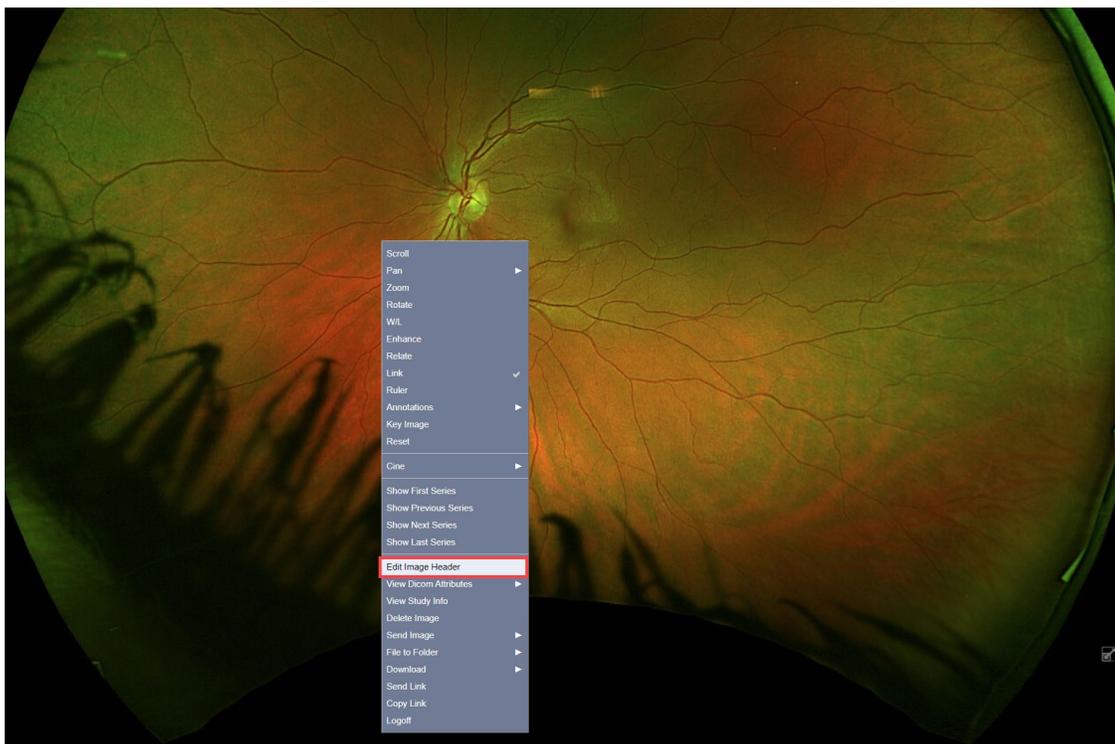

# OptosAdvance™ Changing Laterality of an Image



5 | Choose the correct laterality from the drop-down menu and select **OK** to confirm the change.



6 | A Warning dialog box will appear. select **Continue** to retain your changes.

NOTE: **optomap** images will remain misshapen in OptosAdvance due to the incorrect laterality chosen during device capture.

